ACE X

Atemalkoholtester

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Breathalyzer

Operating Manual – EN

Alcotest

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Alkohol Tester

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Test alcolico

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Alcoholímetro

Manual de instrucciones – ES

Andningsalkoholtestare

Bruksanvisning – SE

Alkomat

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Atemalkoholtester ACE X

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Operating Manual – EN

Author liability

The information contained here has been compiled by ACE Handels- und Entwicklungs GmbH (ACE) with due care and in the belief that it is accurate. However, the author cannot accept any liability for the use of this information or any consequences arising from such use. The author is not liable for any costs or expenses incurred as a result of the use of the information contained in this document. This information is subject to change without notice.

Introduction

Thank you for purchasing our ACE X alcohol tester! This device is an easy-to-use alcohol tester with an electrochemical sensor. The device measures the alcohol concentration in the breath and then calculates the blood alcohol concentration. It may only be used in accordance with the instructions in the operating manual.

This device provides the most accurate test results with a measurement tolerance of $\pm 0.05\%$ (per mille = g/L) at a reference value of 1.00 per mille and an ambient temperature of 25°C.

The measurement unit ‰ (per mille) = default setting in the EU.

The measurement results determined in per mille are displayed with 2 decimal places (e.g., 0.21 ‰) and can be interpreted 1:1 (e.g., the display shows the value 0.36 = 0.36 per mille). Measurement values in % BAC are displayed with 4 digits. The electrochemical sensor integrated in this breathalyzer can be calibrated. To maintain the measurement accuracy of this device and the sensor warranty, we recommend regular calibration. You can send the device to the service address specified at www.ace-technik.com. Upon receipt, ACE will calibrate the device and return it to you for a service fee.

The device may only be used in accordance with the instructions in the user manual. All readings obtained by this breathalyzer are for reference only and are not legally binding. Breath tests should be performed using clip-on mouthpieces (for hygienic reasons and to avoid false readings, a new mouthpiece should be used for each test).

Essential safety guidelines:

Read before use: Please read the entire user manual carefully before using the breathalyzer.

Calibration recommendation: It is recommended that the device be calibrated by a qualified technician every 12 months or after 1,000 tests. Failure to calibrate the device may result in inaccurate measurement results.

Waiting time before testing: Wait at least 15 minutes after eating, drinking, or smoking before performing a test. Otherwise, the result may be inaccurate or invalid.

Residual alcohol in the mouth: Alcohol residues in the mouth can distort the measurement result. Make sure that your mouth is free of residual alcohol.

Contact with water: Never immerse the device in water or other liquids.

Sources of interference from substances: Aromatic beverages or mouth sprays containing alcohol can influence the result. Wait at least 15 minutes or rinse your mouth with clear water before testing.

Cigarette smoke: Do not blow cigarette smoke into the device, as this may damage the sensor.

Legal notice: This device must not be used to decide whether to drive a vehicle or perform other activities after consuming alcohol. Neither the manufacturer nor the distributor is liable for actions taken on the basis of the measurement results.

Remove batteries if not used for a long time: If the device is not used for a long time, remove the batteries. Prolonged storage with batteries inserted may result in inaccurate start-up measurements.

Battery replacement: Replace the batteries as soon as the battery symbol indicates “empty” to maintain measurement accuracy.

Waiting time before retesting: If the measurement result exceeds 0.1% BAC (1.0 g/L) and you retest immediately, the warm-up time will be extended.

Operating temperature: Only perform tests within the temperature range specified in the manual.

Blowing technique: Avoid blowing too hard to prevent saliva from entering the device and damaging the sensor.

Storage conditions: Store the device in a place with moderate temperature and humidity. Room temperature is recommended.

Avoid inhalation: Do not breathe through the device during the test.

Use of the mouthpiece: Use a fresh, clean mouthpiece for each test to ensure measurement accuracy.

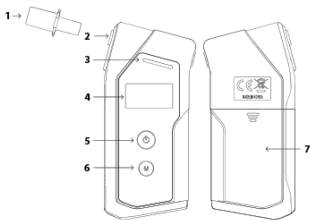
Note on country-specific measurement units

If necessary, please check the current country-specific alcohol limits, which are available, for example, on the website of the Department of Transportation or from automobile clubs.

Scope of delivery

- 1 ACE X digital breathalyzer with electrochemical sensor
- 6 mouthpieces
- 2 AA alkaline batteries (1.5 V)
- 1 hard-shell case
- 1 user manual (multilingual)
- 1 calibration return slip

Description of individual components

1. Mouthpiece
2. Mouthpiece Slot
3. Status LED
4. High-quality OLED display
5. Power button
6. Menu button
7. Battery compartment cover

How to obtain the most accurate result

- A) Wait at least 15 minutes after your last drink before taking the alcohol test. Do not eat, drink, or smoke during this time.
- B) Do not open the case.
- C) Do not store the breathalyzer in a cold, hot, damp, or poorly ventilated place.
- D) Keep the breathalyzer out of the reach of children.
- E) After prolonged storage, perform three "run-in tests" before performing the actual breathalyzer test.
- F) Do not drop the breathalyzer.
- G) Replace the battery when its performance is weak. Reduced battery capacity will lead to incorrect test results.
- H) Do not perform a test in noisy (sound waves interfere with the sensor), drafty, or poorly ventilated places.
- I) After a test result of 1.00 per mille or higher, the sensor needs a longer recovery time. In this case, wait at least three minutes before performing the next test.
- J) Do not perform a test in very cold, very hot, or unventilated, smoky environments.
- K) Store the breathalyzer in such a way that dust and liquids cannot enter it.
- L) Do not blow too hard into the device, as this will damage the sensor (especially if a lot of saliva enters the device when blowing).

Preparing for the test

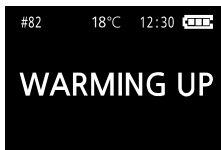
1. Insert 2 AA batteries (alkaline, 1.5 V) into the battery compartment. To do this, carefully remove the battery compartment cover on the back of the device and insert the two batteries. Pay attention to the battery polarity (+/- pole). Do not use batteries with different output voltages (such as manganese or lithium-ion batteries).
2. Insert a mouthpiece into the opening on the side of the device. Make sure that the mouthpiece is securely attached. Use a new mouthpiece for each test subject (for hygiene reasons and to maintain measurement accuracy, as a used mouthpiece may contain residual alcohol).

Performing the measurement

1. Press the power button for 2 seconds to turn on the device. The date and time will appear, as well as the number of days until the next calibration is due.

```
- DATE / TIME
25-02-01 12:30
- NEXT CALIBRATION
365 DAYS
```

2. The device heats up – this process starts automatically:

3. Afterwards, the word "READY TO BLOW" will appear on the display.

4. Performing the test: Place the mouthpiece in your mouth and blow evenly into it. A continuous beep will sound while you are blowing.

Important: Continue blowing until the device automatically completes the test. You will recognize this by a double beep and a mechanical "click-clack" sound. Only then may you stop blowing.

ACE TIP: Blow evenly – imagine you are whistling without making a sound. Do not stop blowing on your own, as this may cause a blow error.

5. In normal mode the measured alcohol level (‰) is displayed after a few seconds (depending on the measurement conditions).

1. Test number
2. Test time
3. Previous result
4. Current result
5. Display time

After displaying the measurement result for 4 seconds, the display switches to a mode in which the following functions are available:

Repeat test: Press the power button briefly to initiate another test

ACE recommends

- Always use a new mouthpiece for each person being tested! ACE accepts no responsibility for any damage that may result from the use of a mouthpiece by multiple people and for multiple tests.
- Always use hygienic, individually packaged ACE mouthpieces. During each test, the inside and outside of the test device comes into contact with human excretions (sweat, saliva, blood, etc.).
- Use gloves when removing the mouthpieces used for a test from the device, especially if the test was performed by other people. ACE Technik (www.ace-technik.com) carries suitable gloves in its product range.

Important information

- If you have eaten, drunk alcohol, or smoked a cigarette, wait at least 15 minutes before starting the measurement.
- For readings below 0.070‰ (OR 0.035 mg/L OR 0.007% BAC – depending on the setting), the test result is displayed as 0.00 (‰ OR mg/L OR % BAC – depending on the setting). This is because at low alcohol levels, other substances may be present in the breath that could influence the result of the breath test.
- This breathalyzer is a high-quality and sensitive measuring device. To maintain the measuring accuracy of the electrochemical sensor, the device should be used regularly. If the device is not used for a long period of time, the sensor may dry out. We recommend using the breathalyzer at least twice a month for breath tests (even if you have not consumed alcohol).
- Changes to the product may result in changes to the procedures described. This may occur without prior notice and without modification of the operating instructions and is not a reason for complaint, return, or price reduction. It is the user's responsibility to recognize any discrepancies between the operating instructions and the actual operation of this breath alcohol tester and to react accordingly.

Only the use of original ACE mouthpieces

- ensures the accuracy of the measurement results
- guarantees hygiene safety
- ensures you receive the full device warranty

ACE warns

Even if the device indicates alcoholization in the "safe" range, this is not a reason to drive a vehicle or perform any other activity that should not be performed under the influence of alcohol after consuming alcohol.

Explanation of symbols:

Error/warning messages

1. Total number of tests performed
2. Current temperature – when Bluetooth is not in use
3. Current time
4. Remaining battery power

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Display	Explanation	Solution
No CAL DATA Contact the service center.	No calibration data: The device does not recognize any calibration data.	Calibration service: Contact your dealer or service center for proper calibration.
OUT OF TEMP RANGE Try again in operating temp range.	Temperature range error: The device is outside the permissible temperature range.	Use within the specified temperature range: Only use the device within the specified temperature range.
CALIBRATION EXPIRED Contact the service center.	Calibration expired: The device has exceeded the recommended calibration interval.	Recalibration: Contact your dealer or service center for recalibration.

TEST LIMIT EXCEEDED Contact the service center.	Number of tests exceeded: The device has exceeded the recommended number of tests.	Recalibration: Contact your dealer or service center for recalibration.
LOW BATTERY Install new batteries.	Battery low: The battery level is too low to take a measurement.	Replacing batteries: Insert new batteries.
BLOW ERROR Repeat the test and blow until the beep stops.	Breath too short: The amount of breath detected is insufficient for an accurate result.	Repeat the test: Perform the test again and follow all instructions carefully.
BLOW ERROR Repeat the test and blow a little weaker.	Blown too hard: The detected breath pressure is too high. The alcohol sensor could be damaged.	Repeat the test: Perform the test again and follow all instructions carefully.
BLOW ERROR Repeat the test without inhaling.	Inhalation detected: Inhalation occurred during blowing.	Repeat the test: Perform the test again and follow all instructions carefully.
EXCESSIVE SALIVA DETECTED# Repeat the test after 20 mins.	Sample with too much saliva: Saliva was detected during the test.	Repeat the test: Wait 20 minutes and perform the test again.
SAMPLING ERROR Contact the service center.	Error in the sampling system: The test cannot be performed due to a problem in the sampling system.	Contact service: If the errors persist, contact your dealer or service center.

Calibration

Calibration is not mandatory, but the accuracy of the integrated sensor will steadily decrease without calibration. Send the device to the address specified on the calibration return slip. We recommend sending it as a parcel so that proof of delivery or a trace request is possible in case it cannot be found. Changes to the product may result in changes to the procedures described. This may occur without prior notice and without modification of the operating instructions and is not a reason for complaint, return, or price reduction. It is the responsibility of the user to recognize any discrepancies between the operating instructions and the actual operation of the ACE X and to respond accordingly.

Notes on batteries

The device to which this operating manual refers contains 2 AA alkaline batteries when delivered. These batteries are not intended to be recharged. Likewise, rechargeable batteries must not be used in the device, as this will damage the device. When the battery is empty, it must not be disposed of in household waste. Used batteries may contain harmful substances that can damage the environment and your health. Please return the batteries/rechargeable batteries to retailers or municipal recycling centers. Returning them is free of charge and required by law. Please only dispose of discharged batteries in the containers provided and tape over the terminals of lithium batteries. Battery recycling is the easiest way to protect the environment – please support it!

The trash cans shown below indicate that old batteries and rechargeable batteries must not be disposed of in household waste. They can be recycled, allowing valuable raw materials such as iron, zinc, and nickel to be recovered.

The abbreviations below the trash cans stand for

- Hg: Battery contains mercury
- Cd: Battery contains cadmium
- Pb: Battery contains lead

ACE WEEE registration number: 89961185

ACE LUCID registration number: DE4358374179837

Technical data*

Sensor	Electrochemical sensor
Accuracy	+/- 0.05‰ at 1.00‰ and 25°C (per mille = default setting) OR +/- 0.005 %BAC at 0.10 % BAC and 25°C OR +/- 0.025 mg/L at 0.50mg/L and 25°C
BAC (blood alcohol concentration)	0.00 to 5.00‰ (per mille = default setting) OR 0.000 to 0.500 % BAC OR 0.00 to 2.50 mg/L
Warm-up time	5 seconds at 0.50 per mille (the higher the per mille value determined, the longer the sensor needs to evaporate the alcohol content measured in the breath)
Test duration/test repetition (readiness)	Within 5 seconds at 0.5‰ or 20 tests per minute at 0.00‰
Device dimensions	60 x 128 x 24mm (H x W x D)
Weight	Approx. 130 g (including 2 AA alkaline batteries)
Power supply	2 AA alkaline batteries (= power for up to 1,500 tests)
Environmental conditions	Operating temperature: -5°C to 50°C
Storage temperature	-5°C to 50°C
Calibration recommendation	After 12 months or 1,000 tests (whichever comes first). You can send the device to the service address listed at www.ace-technik.com for calibration or purchase a replacement sensor there.

*Technical specifications are subject to change without notice due to functional improvements.

MENU OPTIONS

General menu controls

Accessing the menu:

Press and hold the menu button for two seconds during the warm-up process.

Navigating the menu:

Up: Press the power button briefly.

Down: Press the menu button briefly.

Confirming the selection: Press the power button for 2 seconds.

Returning to the warm-up process: Press the menu button for 2 seconds.

Saving the settings:

Press and hold the Power Button for 1 second to save changes directly. In certain menus (TEST MODE, TIME, PIN CODE, CHANGE PIN), you have to press and hold the Menu Button for 1 second to save, and a "Saving Changes" confirmation window will appear. Select "Yes" and then press and hold the Power Button for 1 second to confirm and save. Select "No" and then press and hold the Power Button for 1 second to cancel the change and return to the menu.

TEST RESULT

The test results can be checked under the "TEST RESULT" tab. The screen displays the results of previous tests, including mouthpiece mode (normal mode, mouth alcohol detection mode, and EoB mode), including:

- Test number
- Measurement result
- Test date (YY/MM/DD) and time

The ACE X can print the test result by pressing the power button and the menu button simultaneously.

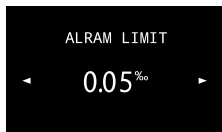
- To display the previous result, briefly press the power button.
- To display the next result, briefly press the menu button.
- To return to the menu list, press and hold the menu button for 2 seconds.

Notes

- The most recent test result is displayed first.
- The oldest test result is automatically deleted when the memory is full.
- The device can store up to 2,000 test results in normal mode, mouth alcohol detection mode, and EoB mode combined.

ALARM LIMIT

The alarm limit can be set in this menu. By default, this is set to 0.5‰ (per mille) or 0.050%BAC (BAC) = same measured value.


Set the alarm threshold for breath alcohol content (BAC). The available settings are:

- OFF or 0.20 to 1.00‰ in 0.10‰ increments (BAC % in 0.010% increments between 0.020 and 0.100% BAC)
- In funnel mode, when set to OFF, 0.20‰ (=0.020% BAC) is automatically used as the limit value
- Factory setting: 0.50‰ (0.050% BAC)

Operation:

- Increase alarm limit: Press the power button briefly
- Decrease alarm limit: Press the menu button briefly
- Save and return to menu: Press and hold the power button for 2 seconds
- Return without saving: Press and hold the menu button for 2 seconds

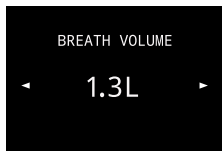BREATH VOLUME

Set the breath volume for the mouthpiece modes (normal mode, mouth alcohol detection mode, and EoB mode). The available options are:

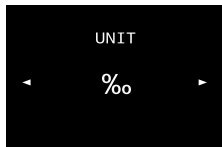
- The breath volume can be set from 0.8 liters to 1.8 liters in 0.1-liter increments.
- Factory setting: 1.2 L

Operation:

- Move cursor up: Press power button briefly
- Move cursor down: Press menu button briefly
- Save and return to menu: Press and hold power button for 2 seconds
- Return without saving: Press and hold menu button for 2 seconds

UNIT

In this menu item, you can set the measuring unit of the device.

The adjustable measuring units are:

- $\mu\text{g/L}$
- mg/L
- ‰
- mg/100mL
- %BAC
- g/210L

The factory setting is: ‰ (per mille)

- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- Save and return to the menu: Hold down the power button for 2 seconds.
- Return without saving: Hold down the menu button for 2 seconds.

BRIGHTNESS

The brightness of the screen can be set in this menu item.

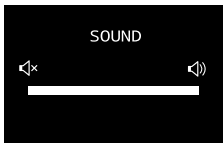
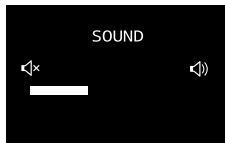
The adjustable value range:

- Level 1–5 (1 = low; 5 = high)
- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- Save and return to the menu: Press and hold the power button for 2 seconds.
- Return without saving: Press and hold the menu button for 2 seconds.

SOUND

The sound volume can be adjusted here.

The volume can be adjusted in this menu.

Adjustable range:

- Level 1–5 (1 = low; 5 = high)
- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- Save and return to the menu: Press and hold the power button for 2 seconds.
- Return without saving: Press and hold the menu button for 2 seconds.

LANGUAGE

The menu language can be selected as follows:

- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- To change the on/off function of each mode, press and hold the power button for 1 second.
- To save the mode change, press and hold the Menu button for 1 second and select "YES" to confirm by pressing and holding the On/Off button for 1 second. If no changes have been made, you can return to the menu list by pressing and holding the Menu button for 1 second.
- To return to the menu list without saving, press and hold the menu button for 1 second.

TIME (TIME/DATE)

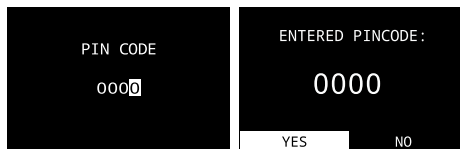
Set the device time.

The time is displayed in YYYY/MM/DD format.

Operation:

- To change digits of time and date, press and hold the Power Button for 1 second.
- To increase the time and date, briefly press the Power Button.
- To decrease the time and date, briefly press the Menu Button.
- To save the mode change, press and hold the Menu Button for 1 second or press the Power Button at the last digit of time (seconds). Select YES by pressing and holding the Power Button for 1 second for confirmation. If no changes were made, you can return to the menu list, by pressing and holding the Menu Button for 1 second.
- To return to the advanced menu list without saving when no changes have been applied, press and hold the Menu Button for 1 second.

PIN CODE

Enter the correct PIN code to access the advanced menu options.

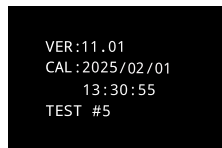
The default PIN code is 0000 and can be changed to a value between 0000 and 9999 in the advanced menu. For more information, see “Changing the PIN code”.

Operation:

- Move to a lower digit of the PIN code: Press and hold the power button for 2 seconds.
- Increase value: Press the power button briefly.
- Decrease value: Press the menu button briefly.
- Save and return to the advanced menu: Once finished, confirm with the power button “Yes” at the fourth (ones) digit of the PIN code.
- Return to the advanced menu without saving: Press the menu button for 2 seconds.

INFORMATION

Further information about the device can be found here.

Display device information such as firmware version, calibration date, and the total number of tests performed.

- **VER:** Current firmware version
- **CAL:** Last calibration date (YYYY/MM/DD) and time
- **TEST:** Number of tests in mouthpiece mode / number of tests in funnel mode

Return to the menu list: Press and hold the menu button for 2 seconds.

ADVANCED MENU:

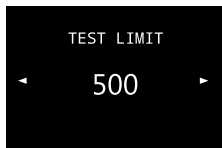
To access the advanced menu, enter the PIN code in the PIN CODE menu item.

For more information, see “PIN CODE”.

The following additional settings are available here:

TEST-LIMIT

In this menu, you can set the maximum number of measurements allowed.

Set the maximum number of tests allowed. The adjustable range is:

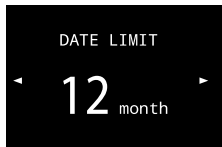
- 50 to 2,000 tests and OFF (no limit)
- ⚙ Factory setting: 500 tests

Operation:

- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- Save and return to the menu: Press and hold the power button for 2 seconds.
- Return without saving: Press and hold the menu button for 2 seconds.

DATE LIMIT

In this menu, you can set the date for the next calibration.

The adjustable value range:

- 1 - 24 months

Factory setting: 12 months

Operation:

- To move the cursor up, briefly press the power button.
- To move the cursor down, briefly press the menu button.
- Save and return to the menu: Press and hold the power button for 2 seconds.
- Return without saving: Press and hold the menu button for 2 seconds.

CHANGE PIN

The PIN code can be set in this menu.

The adjustable value range:

- 0000 ~ 9999
- To change to higher digits of PIN code, press and hold the Power Button for 1 second.
- To increase the number of PIN code, briefly press the Power Button.
- To decrease the number of PIN code, briefly press the Menu Button.
- To save and return to the advanced menu list, press and hold the Power Button for 1 second at the first digit (thousand) of PIN Code or press and hold the Menu Button for 1 second at any digit. Select YES by pressing and holding the Power Button for 1 second for confirmation.
- To return to the advanced menu list without saving when no changes have been applied, press the Menu Button for 1 second.

Warranty and Liability

ACE Handels- & Entwicklungs GmbH (ACE) provides a two-year warranty on material and manufacturing defects for the product to which this warranty and liability statement is attached—in all versions and designations—and a one-year warranty on the sensor. This warranty includes the electronics and the housing and applies to the sensor without sampling components. Components that are inevitably destroyed when activated (e.g., fuses) are not covered by the warranty. The warranty is valid from the date of delivery or from the date of first use (whichever is shorter) if this is specified in the accompanying literature for the ACE product.

WARRANTY AND LIABILITY LIMITATION

1. Product Limitation

ACE disclaims any other express or implied warranty, including merchantability and fitness for a particular purpose. In particular, the warranty does not apply to calibration. Calibration is mandatory after replacing the sensor. If a product requires sensor replacement and/or calibration, the user will receive a cost estimate if necessary. If the user rejects the cost estimate, ACE will return the product unrepaid/uncalibrated and the product will be marked accordingly. If the product does not meet this warranty, the buyer's claim and ACE's obligations are limited exclusively and at ACE's sole discretion to the replacement or repair of the affected goods or to a refund of the purchase price for the goods complained about.

2. Limitation of application

ACE shall in no event be liable for any additional damages, incidental damages, or consequential damages, including, but not limited to, lost profits and loss of use, arising out of the sale, manufacture, or use of the products sold under this disclaimer, regardless of whether such claims arise from contracts, negligence, or other culpable acts.

The product to which this warranty and liability statement is attached, regardless of its version and name, cannot be used in a legally binding manner to determine a person's fitness to drive a vehicle or to perform an action that could endanger themselves or others under the influence of alcohol.

Careful inspection of all products for damage upon receipt, calibration suitable for the buyer's intended use, and proper use, repair, and maintenance in accordance with the instructions in the product literature supplied by ACE are mandatory prerequisites for this warranty. Repair or maintenance work carried out by unqualified personnel will void the warranty, as will the use of non-approved consumables or spare parts. As with all technically sophisticated products, it is a fundamental and mandatory requirement for ACE's warranty that all users of these products are familiar with the use, capabilities, and limitations of the product as described in the relevant product literature.

3. Limitation of advice

The buyer confirms that they have examined the intended use and suitability of the purchased goods for any purpose under their sole responsibility. It is expressly agreed between the parties that any technical or other advice provided by ACE regarding the use of the goods or services was provided free of charge and at the sole risk of the buyer. Therefore, ACE shall not be subject to any obligation or liability arising from the recommendations or their results.

GOOD TO KNOW

Important information about calibration

Any breathalyzer purchased from ACE can also be calibrated by ACE. Of course, we also calibrate devices purchased elsewhere, provided that these devices are identical in construction to those sold by ACE.

You can see the due date for the next calibration on the calibration sticker on your breathalyzer. If the sticker is missing, calibration should take place 6 months after the date of purchase: e.g., date of purchase: May 13 = next calibration on November 13 and then on May 13 of the following year.

You can send the device for calibration to the service address listed at www.ace-technik.com.

Why is calibration necessary?

Calibration is not mandatory, but the accuracy of the integrated sensor decreases steadily without calibration and, in addition, the warranty expires if the calibration intervals are not observed.

Factors that influence measurement accuracy

- The more often the breathalyzer is used, the more the zero point shifts.
- At high alcohol concentrations, the zero point shifts even more.
- The sensitivity of the sensor also decreases in devices that are used little or not at all.

ACE recommends performing a test with the alcohol tester at least once (1x) per week, regardless of whether you have consumed alcohol. To maintain measurement accuracy, the device should also be calibrated regularly.

Please also note the various factors that influence the measurement result

- Amount of alcohol
- Time period of alcohol consumption
- Gender
- Age
- Weight
- Height
- Drinking habits

- Health status
- Liver vitality
- Lung size
- Food intake during alcohol consumption
- Influence of medication

Important information on administering the test and self-assessment

You should wait at least 15 minutes after your last sip of alcohol, as residual alcohol may remain in your mouth and affect the test result. Smoking should also be avoided before a test, as cigarette smoke damages the sensor.

You should use a new mouthpiece for each test – not only for hygiene reasons, but also because a previously used mouthpiece can affect the next test result. Residual alcohol may be present in the mouthpiece, which could affect the new test result if the same mouthpiece is used again (for example, a sober person might be shown to be intoxicated).

If “Blow” appears on the display, the breath sample was insufficient or it was ended too early. Blow steadily for approximately 4 seconds during your breath sample, ideally as if you were whistling silently. The breath sample is complete as soon as you hear two beeps.

After a test, the alcohol molecules are stirred up in the lungs and it takes a while for them to collect again in the capillaries at the base of the lungs. The tester wants to test the most concentrated air.

All readings determined with this breathalyzer are only guidelines and not legally binding.

ACE note:

No breathalyzer is valid in court. (Exception: a few stationary, very expensive devices that measure using infrared!) The only test that is and remains valid in court is the blood test.

Why perceived alcohol concentration and the measurement result often do not match

There are many different factors that influence the reading in a breathalyzer test. These include gender, age, height, and weight, as well as body fat percentage and body fluid content.

In addition, the general physical and mental state and the individual's current state of mind must also be taken into account. Furthermore, the type of meal before the test (type of contents and level of stomach fluid) and the use of medication can influence the reading.

Other influencing factors include, of course, the type of beverage (percentage of alcohol in the beverage, as well as its carbon dioxide and sugar content), the speed of consumption, and the individual duration of alcohol buildup and breakdown in the body, which depends on the test subject.

Some people quickly experience elevated blood alcohol levels, while for others, it takes longer

to detect intoxication. As a rough guide, we can say: approximately 40-50% of the alcohol consumed is in the body within 15 minutes, after 30 minutes it is 65%, and after 70 minutes at the latest, the previously consumed alcohol is fully absorbed. It should also be noted, however, that some people – partly due to genetic factors – break down their blood alcohol levels quickly, whereas others continue to have elevated blood alcohol levels even after a longer period of time.

LEGAL INFORMATION

This product meets the essential requirements of the following EU directives:

- 2014/30/EU (EMC Directive)
- 2014/35/EU (Low Voltage Directive – LVD)
- 2011/65/EU (RoHS Directive)

Conformity has been assessed based on the following harmonized standards

- EN IEC 61326-1:2021 (EMC – Electrical Measuring Instruments)
- EN 61010-1:2010/A1 (Safety – Electrical Measuring Instruments)
- EN 62321-4:2014 (RoHS – Substance Evaluation)

The full EU Declaration of Conformity is available at: www.ace-technik.com

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