ACE AL6000

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Important Notes – EN

On the Calibration

Each breathalyzer which you buy from ACE Instruments can also be calibrated by ACE Instruments. Of course, we also calibrate devices which were bought from other contractors, as long as these devices are identical in construction to the devices sold by ACE Instruments.

You can see the due date of the next calibration on the calibration sticker placed on your breathalyzer. If the sticker is missing, the calibration is to be done 6 months after the purchase date: e.g. purchase date: 13.05. = next calibration on the 13.11. and the next on the 13.05. of the following year.

For a calibration service you can send the device to the service address specified at www.alkomat.net.

Why is a calibration necessary?

The calibration is not mandatory, but the accuracy of the sensor installed in the breathalyzer will decrease steadily without a calibration and additionally guarantee expires, if you don't pay attention to the calibration ranges.

Which factors influence the measurement accuracy

- The more often the device is used the more the zero point is influenced.
- The higher the alcohol concentration is, the more the zero point is relocated.
- Devices which are used rarely have a decreased sensibility, too.

ACE Instruments recommends using the breathalyzer at least once (1x) a week independently of having drunken alcohol or not. To maintain the measurement accuracy, the breathalyzer should also be calibrated periodically.

Please also be aware of the several factors of influence on the measuring result

- Alcohol amount
- Period of the consumption of alcohol
- Gender
- Age
- Weight
- Height
- Drinking habits
- Health state
- Vitality of the liver
- Size of the lung
- Admission of food during the consumption of alcohol
- Influence of medicine

Important notes on test execution and self-assessment

After the last sip of alcohol you should wait at least 15 minutes, as there may still be residual alcohol in the oral cavity, which influences the measurement result. Also smoking should be avoided before a test, as cigarette smoke harms the sensor.

For each test you should use a new mouthpiece – not only for hygienic reasons, but also because an already used mouthpiece can influence the next measurement result. The mouthpiece could contain residual alcohol, which would affect the new test result when the same mouthpiece is used again (this for example could show an alcoholization of a person who hasn't drunken any alcohol).

If „FLOW“ or “Flow” or „Flo“ appears on the display, the given breath sample was insufficient or ended too soon. Blow into the moutpiece for about 4 seconds constantly (without a break) – preferably as if you were whistling without a sound.

If the same person performs 2 tests consecutively, then wait at least 3 minutes between the two tests, as a too short waiting time may cause different measurement results. The air exhaled from the lungs is, after a test, no longer enriched with the same alcohol content as in the previous test. Therefore, an immediate second test will indicate a lower test result.

All measured values determined with this breathalyzer are always indicative and not legally binding.

Why the estimated alcoholization and the measurement result in most cases cannot match

An alcohol test is affected by a lot of different factors which also affect the test result. Beside the gender, age, body size and body weight, also the body fat percentage and the amount of body fluid have a big influence.

In addition, the general physical and mental state or the current daily constitution have to be taken into consideration. Furthermore, the meals eaten before a breath test (type of content and level of the stomach) and the intake of medication may influence the measurement.

Other influencing factors are, of course, the type of drink (percentage of alcohol content in the beverage and its carbon dioxide and sugar content), the drinking rate and the individual subject's duration of alcohol build-up or degradation in the body.

Some people quickly have a high alcohol content in the blood, with other people it takes longer until an alcoholization is detected. About 40-50% of the consumed alcohol is in the body within 15 minutes, after 30 minutes it is 65% and after 70 minutes the previously consumed alcohol is 100% absorbed in the body. However, it should also be noted that some people – partly due to genetic factors – quickly reduce the alcohol content in the blood, whereas others still have an increased alcohol content in the blood after a longer period of time.

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Operating manual – EN

Author's Liability

All information herein was written by the ACE Handels- und Entwicklungs GmbH (ACE Instruments) conscientiously and under the assumption of correctness. Hence, the author cannot be held liable for use of any kind whatsoever and/or any ensuing consequences. Furthermore, the author is not liable for any costs or expenses for damage that result from any information in this document. Any information is subject to change without prior notice.

Introduction

Thank you for purchasing our ACE AL6000 breathalyzer! It measures the alcohol concentration in human breath using semiconductor sensor and calculates the blood alcohol content. The test results are very reliable and have a tolerance of $\pm 0.10\text{‰}$ (per mill = g/L) with reference to 0.50‰ at 25°C ambient temperature. The measurement results are displayed as a 3-digit number with 2 decimals and are interpretable 1: 1 (for example the display shows the value $0.36 = 0.36$ per mill).

The sensor inside the device can be calibrated. To maintain the measurement accuracy of this device and the sensor warranty, it should be calibrated regularly. For a calibration service, you can send the device to the service address specified at www.alkomat.net.

The device must only be used pursuant to the instructions mentioned in this manual. All measured values determined by this breathalyzer are only a guide and not legally binding. Breath tests should be carried out using clip-on mouthpieces (one new mouthpiece should be used per test for reasons of hygiene and to avoid falsified readings).

Note to Country-specific Measurement Units

Customers in the EU receive the breathalyzer with ‰ (per mill) as default setting, in the UK the default setting is mg/L, and in the USA, it is % BAC. In the chapter „Technical data“ you will find information about the accuracy concerning the different measurement units. Please check e.g. the website of the Department of Transportation, the website of the Ministry of Transport, or automotive clubs to become informed about the current alcohol limits in your country.

Scope of Delivery

- 1 Digital ACE AL6000 Breathalyzer with semiconductor sensor
- 2 AA batteries (1.5 V)
- 5 Mouthpieces
- 1 Soft bag with carabiner
- 1 Hand strap
- 1 Quick Start Guide (in German & English)
- 1 Manual (multilingual)

Label of Individual Components

1. Mouthpiece and Mouthpiece Slot

2. Digital LED Display (3 digits with 2 decimals in the unit ‰ and mg/L)

3. Power Button

4. Display of status readiness (READY), warning (WARN) and status of battery (BATT LOW)

5. Air Outlet (always leave this part unblocked)

6. Battery Compartment Cover (at the bottom of the device)

Tips to Receive the Most Accurate Results

- A) Wait at least 15 minutes after your last sip of alcohol. Do not eat, drink or smoke during that time.
- B) Do not open the casing.
- C) Do not keep the breathalyzer in cold, hot, moist, or badly ventilated places..
- D) Keep the breathalyzer out of the reach of children.
- E) Conduct three inlet tests after long storage before conducting the actual alcohol test.
- F) Do not drop the breathalyzer.
- G) Change the battery if it is depleted. Diminished battery capacity leads to incorrect test results.
- H) Do not conduct tests in loud (sound waves disrupt the sensor), drafty, or poorly ventilated places.
- I) After a test result of 1.00 per mill (per mill = EU setting) / 0.10 % BAC (% BAC = US setting) / 0.50mg/L (mg/L = UK setting) or higher the sensor needs rest time. In this case wait at least three minutes before starting the next test.
- J) Do not conduct tests in very cold, very hot, or an unventilated, smoky ambience.
- K) Store the breathalyzer in a way to avoid penetration by dust or liquids.
- L) Blowing too hard may cause damage to the sensor as saliva flows into it.

Preparation for the Test

- 1. Push the battery cover (at the bottom of the device) sideways and open it. Install the 2 AA batteries (delivered together with the device) in the battery compartment while taking note of the battery polarity (+/-). The polarity is displayed on the inner side of the battery compartment cover. Close the battery cover again.
- 2. Place a mouthpiece on the side opening for attaching the mouthpieces (= the opening on the right side of the front panel). Make sure that the mouthpiece is tight. Use a new mouthpiece for each subject (for hygienic reasons and to maintain measurement accuracy, as a used mouthpiece may contain residual alcohol inside).

Note: The ACE AL6000 provides the possibility to test in an active/normal and in a passive mode. In the active mode you have to use mouthpieces for a test, in the passive mode no mouthpieces are necessary.

Conducting the Measurement
Active/Normal Mode Operation

- 1. Press and hold the power switch to turn the unit on. You alternately can see the cumulative test count, *FSF* and *nor*.
- 2. Release the power switch with test counter when *nor* sign is displayed, the device is now in active mode operation. Alternatively, you can turn on the unit in active mode by holding down the start button for only 2 seconds.
- 3. There will be an automatic countdown from *234* to *0* as the unit warms up. At the end of the countdown you will hear two beep tones, the LED "READY" will blink and you will see *0* on the display – the device is ready.
- 4. If you don't start with the breath sample within a few seconds the device will put itself off automatically.
- 5. Secure your lips around the mouthpiece and blow sharply and steadily into the sensor opening near the top of the unit for at least 5 seconds, during this you will hear a long beep.
- 6. If the breath sample is too weak you will see *Flo* on the display, then the device shuts itself off.
- 7. If the breath sample is successful you will hear two beep tones as soon as the breath sample is sufficient. You will see the digit *0* moving left to right in the display as the device is analyzing the sample.
- 8. When the analysis is complete, you will see the blood alcohol concentration (BAC) results displayed in per mill.
- 9. The result will be displayed for approximately 15 seconds, then the device will put itself off.

Interpretation of the Results in Active Mode Operation

Display	Meaning	Alcoholization
0.00‰ to 0.10‰	Safe range	Light/no alcohol intake
0.11‰ to 0.40‰	Moderate range	Increased alcohol intake
0.41‰ and more	Warning range	Ability to drive may be impaired (the LED-display "WARN" is shown)

Passive Mode Operation

- 1. Press and hold the power switch to turn the unit on. You can see the cumulative test count, *FSF* and *nor*.
- 2. Release the power switch when *FSF* sign is displayed, the device is now in passive mode operation.
- 3. There will be an automatic countdown from *234* to *0* as the unit warms up. At the end of the countdown you will hear two beep tones and see *FSF* on the display, the device is ready now. In the passive mode the device shuts itself off automatically after a longer period of time (in seconds) than in the active mode.
- 4. Blow sharply and steadily into the sensor opening near the top of the unit for at least 1 second, during this you will hear a constant beep tone. You will hear two bleeps when the device has finished taking the breath sample.
- 5. If the breath sample is successful you will see the digit „0" moving left to right in the display as the device is analyzing the sample.
- 6. When the analysis is completed you will see the blood alcohol concentration (BAC) results displayed in per mill.
- 7. In the passive mode you can immediately do another test when you see *FSF* in the display after the unit has shown the measurement for the previous test.

Interpretation of the Results in Passive Mode Operation

In this modus the usage of mouthpieces is optional. You can use this modus if a person has drunken too much alcohol or is badly injured and therefore is not able to provide an active breath sample. The ACE AL6000 only needs a weak breath sample and shows the results in 3 approximate ranges.

Concentration	Meaning	Alcoholization	Display
0.00‰ to 0.10‰	Safe range	Light/no alcohol intake	<i>0</i>
0.11‰ to 0.40‰	Moderate range	Increased alcohol intake	<i>Lo</i>
0.41‰ and more	Warning range	Ability to drive may be impaired (the LED-display "WARN" is shown)	<i>Hi</i>

Note: When conducting a test in this modus the ACE AL6000 can be contaminated with human secretion. Please clean the device with a disinfectant before starting a new breath sample.

ACE Instruments Recommends

- Always use a new mouthpiece for each person and test that is conducted. ACE Instruments assumes no liability for any damages that may arise when several people use the same mouthpiece for multiple tests.
- Only use the provided individually packaged and hygienic mouthpieces. During each test the breathalyzer's internal and external parts come into contact with human secretion (saliva, blood, sweat, etc.)!
- Use safety gloves when pulling out a mouthpiece used by another person! You can buy appropriate gloves at ACE Technik (www.ace-technik.de).

IMPORTANT NOTICE

- WHEN YOU HAVE EATEN, DRUNK ALCOHOL OR SMOKED A CIGARETTE, WAIT AT LEAST 15 MINUTES BEFORE STARTING THE MEASUREMENT. ANY TEST CONDUCTED PRIOR TO THE EXPIRATION OF THE RECOMMENDED WAITING PERIOD WILL RESULT IN INACCURATE RESULTS.
- IF THE TEST RESULT IS BELOW 0.10‰ (PER MILL = g/L) OR 0.01% BAC (US setting) OR 0.05mg/L (UK setting) THEN THE DEVICE SHOWS 0.00 (‰ OR % BAC OR mg/L – depending on the setting) ON THE DISPLAY. THIS HAPPENS BECAUSE AT A LOW RANGE OF ALCOHOL CONCENTRATION IT IS POSSIBLE TO MEASURE OTHER MATERIALS IN YOUR MOUTH.
- This breathalyzer is a high quality and sensitive measurement device. To maintain the accuracy of the sensor, it should regularly be used. If the device is not used for an extended period, the sensor can dry out. ACE Instruments recommends using the breathalyzer at least 2 times per month (even if you have not been drinking alcohol).
- Due to changes in the product, the procedures stated can change. This can take place without prior notice or adaptation of the operating manual and is no cause for complaint, return or price reduction. The user is responsible for recognizing discrepancies between the operating manual and the actual handling of the breathalyzer and must react accordingly.

Only the Usage of Original ACE Instruments Mouthpieces

- Guarantees test result accuracy
- Guarantees 100% hygiene
- Guarantees full warranty of this device

ACE INSTRUMENTS ADVISES
 EVEN IF YOU GET A RESULT MEANING „SAFE RANGE“, YOU SHOULD NEVER DRIVE AFTER CONSUMING ALCOHOL OR DO ANY OTHER ACTIVITIES THAT ARE SEVERELY IMPAIRED BY ALCOHOL CONSUMPTION AND MIGHT ENDANGER YOUR HEALTH AND SAFETY OR THE HEALTH AND SAFETY OF OTHERS.

Display Indications


Display	Meaning	Action
<i>BAT</i>	The testing cannot be performed because the battery is low.	Replace with new batteries. Be aware of the polarity when inserting the batteries (+/- pole).
<i>CAL</i>	The device should be calibrated (to maintain measurement accuracy and sensor warranty).	Please send the device in for calibration or purchase a new sensor. You can send the device for calibration to the service address given at www.alkomat.net or order a new sensor in the shop.
<i>Flo</i>	The sampling volume or blow power is not enough to analyze the breath sample.	Switch the device on again and blow steadily, constantly and long enough into the mouthpiece or mouthpiece opening (depending on the selected test mode). It is best to blow as if you were whistling without a sound.
<i>Out</i>	If no breath sample was given within 3 minutes, the device turns off automatically. This also happens after 1 minute without using the device.	Press power button to restart the countdown cycle and retest.
<i>SEN</i>	The sensor is defective.	Purchase a new sensor or send the instrument to ACE Instruments (www.alkomat.net) for service.

Change of the Sensor


1. Loosen the screw on the top of the device but don't pull it out.
2. Carefully remove the cover and then the sensor module like it is shown on the two pictures on the right side. Do NOT detach the cable from the body!
3. Remove the cable from the slot underneath the sensor module and pull the old sensor off from the metal blank.

4. After preparing a new calibrated or new sensor module insert the cable into the new sensor. Clip on the new sensor on the metal blank.
5. Insert the sensor into the frame.
6. The two pictures on the right show the correct position of the sensor.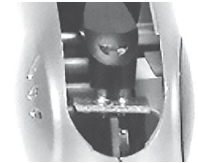

7. Re-attach the cover on the top of the body. It should snap in fairly easily.
8. The front of the cover has to fit exactly into the body. If the pins in front of the sensor don't fit, please carefully press them back into the body.
9. To finish the process, size the screw again.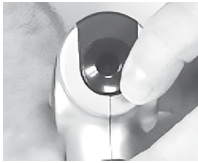

Calibration
 To maintain the accuracy of the breathalyzer, ACE Instruments recommends a calibration after 500 tests or every 6 to 12 months (depending which incident happens first). For a calibration service you can send the device to ACE Instruments (see www.alkomat.net). The calibration is not mandatory, but the accuracy of the breathalyzer will decrease steadily and additionally the warranty for the sensor expires.
 We recommend sending it as a parcel, so that in case of loss an inquiry or proof of delivery is possible.


Battery notices

The device associated with this operating manual contains 2 AA Alkaline batteries (1.5 V) in delivery condition. These batteries are non-rechargeable. The use of a rechargeable battery is not possible with this device and can damage it. If the included batteries are depleted, they must not be thrown away into household waste. Used batteries may contain harmful substances that can cause damage to the environment or your health. Please return all depleted batteries to electrical appliance stores, who collect them for free.

Battery exchange

To replace the battery, press the small lever on the bottom of the unit and pull up the battery compartment cover. Replace the discharged batteries with new ones. Be aware of the polarity when inserting the batteries (+/- pole). Close the battery compartment cover again. In the EU, the return of used batteries is mandatory by law. All depleted and rechargeable batteries will be recycled. This way valuable resources like iron, zinc and nickel can be retrieved and reused. Battery recycling is the easiest lesson in environmental protection – so please support it!

The shown garbage bins mean that batteries or rechargeable batteries must not be thrown into the household waste. The signs below the garbage bins are acronyms for the chemical substances inside the battery:

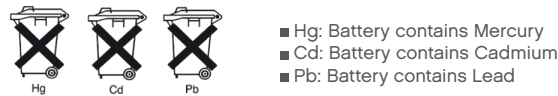Technical data*

Sensor	Semiconductor sensor
Accuracy	+/- 0.10‰ (per mill) with reference to 0.50‰ at 25°C (per mill = default setting) OR +/- 0.01% BAC (Blood Alcohol Concentration) with reference to 0.05 % BAC at 25°C OR +/- 0.05mg/L with reference to 0.25mg/L at 25°C
Detection Range	0.00 up to 4.00‰ (per mill) OR 0.00 up to 0.40% BAC (USA) OR 0.00 up to 2.00mg/L (UK)
Warm-up Time	Within max. 30 seconds at 0.50‰, this may vary depending on the measured value of per mill / % BAC / mg/L
Dimension	125 x 65 x 38mm (L x W x H)
Weight	117g (batteries included)
Power Supply	2 pieces of 1.5V AA Alkaline batteries (= power for up to 500 tests)
Working Temperature	5°C to 40°C
Storage Temperature	0°C to 40°C
Calibration	Recommendation: Every 6 to 12 months or after 500 tests. For a calibration service you can send the device to ACE Instruments (see www.alkomat.net).

* The technical data are subject to change without prior notice for functional improvements.

Warranty and liability

ACE Handels- und Entwicklungs GmbH (ACE Instruments) provides a warranty of two years after purchase for material and processing errors of the product (see attached warranty and liability declaration) in all versions and designations; and a warranty of one year for the sensor. This warranty includes the electronics as well as the casing and sensor without sampling components. Components that inevitably degrade over time (such as fuses) are not part of this warranty. The warranty is valid from the date of delivery or from the date of first activation (always the shorter period), if not stated otherwise in the accompanying ACE Instruments product literature.

Limitation of warranty and liability

1. PRODUCT DESCRIPTION

ACE Instruments rejects any further expressly stated or implied warranty, including the marketability and suitability for a determined purpose. This warranty especially excludes calibration. After changing the sensor, calibration is absolutely necessary. If an exchange of the sensor or a calibration is required for a product, the user may receive a cost estimate. Should the user reject the estimate, ACE Instruments will send the product back unrepaired/not calibrated and the product will be labelled accordingly. Should the product not fulfil the warranty, the claim of the buyer and the obligations of ACE Instruments are limited exclusively to the discretion of ACE Instruments for exchange, repair of the affected goods or reimbursement of the purchasing price for the faulty merchandise.

2. APPLICATION RESTRICTION

ACE Instruments is by no means liable for additional, incidental or consequential damage such as, among others, lost profit and downtime that result from selling, producing or using products that are part of this application restriction. This is independent of any claims emanating from, for instance, contracts, negligence or other culpable actions. The product, which is accompanied by this warranty and liability declaration (no matter the version or designation), in particular cannot be used in a legally binding manner to determine one's capability to operate vehicles or execute action that could lead to personal injury or injury to others under the influence of alcohol.

The thorough check for damage of all products upon receipt, calibration suitable for the given use by the buyer and the proper use, maintenance, and service according to the instructions of the accompanying ACE Instruments product literature are mandatory prerequisites of this warranty. Maintenance and service work performed by non-qualified persons as well as the use of non-approved consumables or spare parts results in the voiding of this warranty. As for all technically challenging products, it is a fundamental prerequisite and mandatory for ACE Instruments warranty that all users of these product are familiar with the properties and restrictions of the devices as stated in the respective literature.

3. ADVICE RESTRICTION

The buyer is solely responsible and confirms that they have checked the intended use and suitability of purchased goods for any purpose. Between the two parties it is expressly agreed that all technical or other advice given by ACE Instruments regarding the use of goods or services is free of charge and made under risk to the buyer exclusively. Therefore, no obligation or liability can result to ACE Instruments from any recommendation or their consequences.

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ACE Handels- & Entwicklungs GmbH